


Step 1

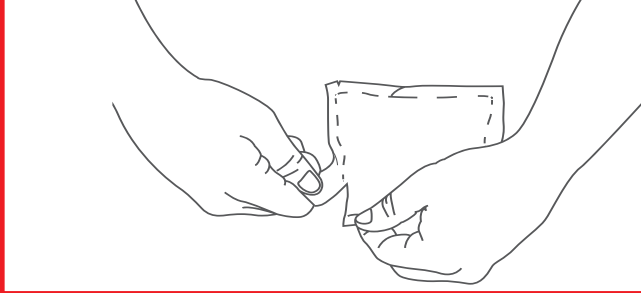


Open the pack and take out WoundClot Hemostatic Gauze.

Step 2




Hold by pointed end.

Step 3

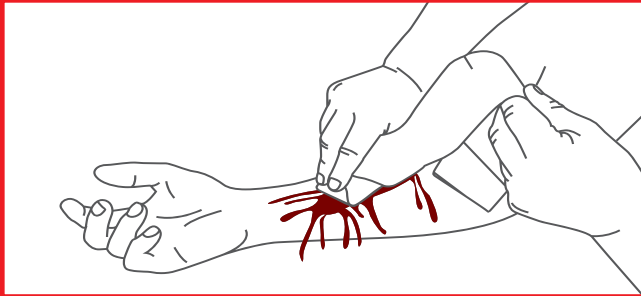


Place the gauze gently into the wound and let the gauze soak in blood while pressing lightly to bring the gauze in contact with the bleeding area.

Step 4

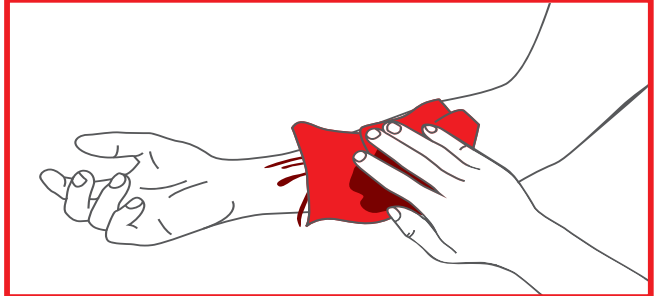

If required, place a sponge or pressure bandage gently on top of WoundClot Hemostatic Gauze to secure it in place until further treatment is given.

Step 5

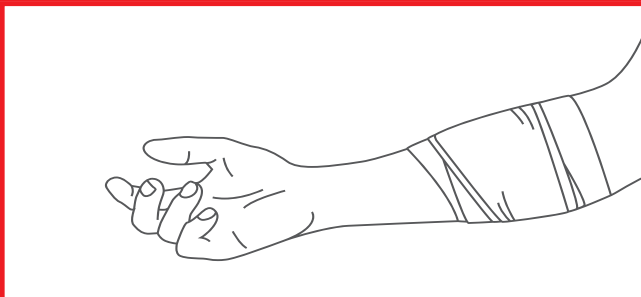

If bleeding persists, remove the dressing and repeat steps 1-4.

Removal

Once Hemostasis is achieved, remove by saturating the gauze with water or saline solution, then gently remove any excess gauze material away from the wound.